en	Instructions for use/Technical description Accessories for sterilizer baskets (silicone/plastic)
USA	Note for U.S. users This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at www.aesculapusa.com . If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.
de	Gebrauchsanweisung/Technische Beschreibung Zubehör für Sterilisieriebschalen (Silikon/Kunststoff)
fr	Mode d'emploi/Description technique Accessoires pour tamis de stérilisation (silicone/plastique)
es	Instrucciones de manejo/Descripción técnica Accesorios para bandejas de esterilización (silicona/plástico)
it	Istruzioni per l'uso/Descrizione tecnica Accessori per cestelli per sterilizzazione (silicone/plastica)
pt	Instruções de utilização/Descrição técnica Acessórios para taças de rede de esterilização (silicone/plástico)
nl	Gebruiksaanwijzing/Technische beschrijving Toebehoren voor sterilisatiezeefschalen (silicone/kunststof)
da	Brugsanvisning/Teknisk beskrivelse Tilbehør til sterilisatorsigter (silikone/plast)
sv	Bruksanvisning/Teknisk beskrivning Tillbehör till steriliseringsstrådkorgar (silikon/plast)
fi	Käyttöohje/Tekninen kuvaus Lisätarvikkeet sterilointisihikoreihin (silikoni/muovi)
lv	Lietošanas instrukcijas/tehniskais apraksts Sterilizācijas sieta trauku piederumi (silikons/plastmasa)
lt	Naudojimo instrukcija/techninis aprašas Priedas tinkliniam sterilizavimo kiuvetėi (iš silikono/plastiko)
ru	Инструкция по применению/Техническое описание Принадлежности для стерилизационных сетчатых корзин (силикон/пластик)
cs	Návod k použití/Technický popis Příslušenství pro sterilizační síta (silikon/plast)
pl	Instrukcja użytkowania/Opis techniczny Aksesoria do koszy sterylizacyjnych (silikon / tworzywo sztuczne)
sk	Návod na použitie/Technický opis Príslušenstvo pre sterilizačné sieťové misky (silikón, plast)
hu	Használati útmutató/Műszaki leírás Tartozék sterilizáló szűrőtálcahoz (szilikon/műanyag)
sl	Navodila za uporabo/Tehnični opis Dodatki za mrežaste košare za sterilizacijo (silikon/plastika)
hr	Upute za uporabu/Tehnički opis Pribor za rešetkaste posude za sterilizaciju (silikon/plastika)
ro	Manual de utilizare/Descriere tehnică Accesorii pentru tăvile de filtrare sterilizante (silicon/material plastic)
bg	Упътване за употреба/Техническо описание Принадлежности за кошници за стерилизиране (силикон/пластмаса)
tr	Kullanım Kılavuzu/Teknik açıklama Sterilizatör süzgeç tepşirler için aksesuarlar (silikon/plastik)
el	Οδηγίες χρήσης/Τεχνική περιγραφή Πρόσθετα εξαρτήματα για διάτρητους δίσκους αποστείρωσης (σιλκόνη/πλαστικό)

1. About this document

1.1 Scope

These instructions for use apply to Aesculap accessories made of silicone/plastic or having silicone/plastic components for sterilizer baskets.

- ▶ For article specific instructions for use and material compatibility and lifetime information, see B. Braun eIFU at eifu.bb Braun.com

1.2 Safety messages

Safety messages make clear the dangers to patient, user and/or product that could arise during the use of the product. Safety messages are labeled as follows:

⚠ WARNING

Indicates a possible threat of danger. If not avoided, minor or moderate injury may result.

⚠ CAUTION

Indicates a possible threat of material damage. If not avoided, the product may be damaged.

2. Clinical use

2.1 Areas of use and limitations of use

2.1.1 Intended use

Accessories made of silicone/plastic or having silicone/plastic components for sterilizer baskets are used by OP and AEMP personnel and are intended for repeated use. They serve to hold goods to be sterilized in position in the sterilizer basket as much as possible during the sterilization process, transport and storage.

2.1.2 Indications

Note

The manufacturer is not responsible for any use of the product against the specified indications and/or the described applications.

For indications, see Intended use.

2.1.3 Contraindications

No known contraindications.

2.2 Safety information

2.2.1 Clinical User

General safety information

To prevent damage caused by improper setup or operation, and to not compromise the manufacturer warranty and liability:

- ▶ Use the product only according to these instructions for use.
- ▶ Follow the safety and maintenance instructions.
- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- ▶ Store the new or unused products in a dry, clean and safe place.
- ▶ Prior to use, check that the product is in good working order.
- ▶ Keep the instructions for use accessible for the user.

Note

The user is obligated to report all of the severe events in connection with the product to the manufacturer and the competent state authorities in which the user is located.

2.2.2 Sterility

The product is delivered in an unsterile condition.

- ▶ Clean the new product after removing its transport packaging and prior to its initial sterilization.

2.3 Application

⚠ WARNING

Risk of injury and/or malfunction!

- ▶ Prior to each use, inspect the product for loose, bent, broken, cracked, worn, or fractured components.
- ▶ Always carry out a function test prior to each use of the product.

Note

As required, elements made of silicone can be trimmed to have the desired shape.

3. Validated reprocessing procedure

3.1 General safety information

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for sterile processing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note

Mechanical reprocessing should be favored over manual cleaning as it gives better and more reliable results.

Note

Successful processing of this medical device can only be ensured if the processing method is first validated. The operator/sterile processing technician is responsible for this.

Note

If there is no final sterilization, then a virucidal disinfectant must be used.

Note

For up-to-date information about reprocessing and material compatibility, see B. Braun eIFU at eifu.bb Braun.com. The validated steam sterilization procedure was carried out in the Aesculap sterile container system.

3.2 General information

Dried or affixed surgical residues can make cleaning more difficult or ineffective and lead to corrosion. Therefore the time interval between application and processing should not exceed 6 h; also, neither fixating pre-cleaning temperatures >45 °C nor fixating disinfecting agents (active ingredient: aldehydes/alcohols) should be used.

Excessive measures of neutralizing agents or basic cleaners may result in a chemical attack and/or to fading and the laser marking becoming unreadable visually or by machine for stainless steel.

Residues containing chlorine or chlorides e.g. in surgical residues, medicines, saline solutions and in the service water used for cleaning, disinfection and sterilization will cause corrosion damage (pitting, stress corrosion) and result in the destruction of stainless steel products. These must be removed by rinsing thoroughly with demineralized water and then drying.

Additional drying, if necessary.

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Optical changes of materials, e.g. fading or discoloration of titanium or aluminum. For aluminum, the application/process solution only needs to be of pH >8 to cause visible surface changes.
- Material damage such as corrosion, cracks, fracturing, premature aging or swelling.
- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surfaces and could cause corrosion.
- ▶ Further detailed advice on hygienically safe and material-/value-preserving reprocessing can be found at www.a-k-i.org, link to "AKI-Brochures", "Red brochure".

3.3 Limitations on reuse

Influences of the reprocessing which lead to damage to the product are not known.

A careful visual and functional inspection before the next use is the best opportunity to recognize a product that is no longer functional, see Inspection.

3.4 Preparations at the place of use

- ▶ If applicable, rinse non-visible surfaces preferably with deionized water, with a disposable syringe for example.
- ▶ Remove any visible surgical residues to the extent possible with a damp, lint-free cloth.
- ▶ Transport the dry product in a sealed waste container for cleaning and disinfection within 6 hours.

3.5 Cleaning/Disinfection

3.5.1 Product-specific safety information on the reprocessing method

Risk of infection or damage to the medical devices due to insufficient or inappropriate reprocessing!

The reprocessing procedure described below applies only to unloaded trays.

- ▶ For loaded trays, read and follow the instructions for reprocessing the products in their respective instructions for use.

Damage to or destruction of the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!

- ▶ Use cleaning agents and disinfectants according to the manufacturer's instructions.
- ▶ Observe specifications regarding concentration, temperature and exposure time.
- ▶ Use suitable cleaning/disinfecting agents if the product is put away in a wet condition. To prevent foaming and degradation of the efficacy of the process chemicals: Prior to mechanical cleaning and disinfection, rinse the product thoroughly with running water.

3.5.2 Validated cleaning and disinfection procedure

Validated procedure	Specific requirements	Reference
Mechanical alkaline cleaning and thermal disinfection	▶ Place the product in the tray as appropriate for cleaning (to avoid rinsing blind spots).	Chapter Mechanical cleaning/disinfection and subsection: ■ Chapter Mechanical alkaline cleaning and thermal disinfection

3.6 Mechanical cleaning/disinfection

Note

The cleaning and disinfection device must be of tested and approved effectiveness (e.g. FDA approval or CE mark according to DIN EN ISO 15883).

Note

The cleaning and disinfection device used for processing must be serviced and checked at regular intervals.

3.6.1 Mechanical alkaline cleaning and thermal disinfection

Machine type: single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemical/Note
I	Pre-rinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	FD-W	■ Concentrate, alkaline: - pH = 13 - <5 % anionic surfactant ■ 0.5 % working solution - pH = 11*
III	Intermediate rinse	>10/50	1	FD-W	-
IV	Thermal disinfecting	90/194	5	FD-W	-
V	Drying	-	-	-	According to the program for cleaning and disinfection device

D-W: Drinking water

FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)

*Recommended: BBraun Helimatic Cleaner alkaline

- ▶ Check visible surfaces for residues after mechanical cleaning/disinfecting.

3.7 Inspection

- ▶ Allow the product to cool down to room temperature.
- ▶ Dry the product if it is wet or damp.

3.7.1 Visual inspection

- ▶ Ensure that all contamination has been removed. In particular, pay attention to mating surfaces, hinges, shafts and recessed areas.
- ▶ If the product is dirty: repeat the cleaning and disinfection process.
- ▶ Check the product for damage, e.g. corroded, loose, bent, broken, cracked, worn or severely scratched and fractured components.
- ▶ Check the product for missing or faded labels.
- ▶ Check the surfaces for rough spots.
- ▶ Check the product for burrs that could damage tissue or surgical gloves.
- ▶ Check the product for loose or missing parts.
- ▶ Immediately put aside damaged or inoperative products and send them to Aesculap Technical Service, see Technical service.

3.7.2 Functional test

- ▶ Check that the product functions correctly.
- ▶ Check for compatibility with associated products.
- ▶ Immediately put aside inoperative products and send them to Aesculap Technical Service, see Technical service.

3.8 Steam sterilization

- ▶ Make certain that all external and internal surfaces of the product will be exposed to the sterilizing agent.
- ▶ Validated sterilization process
 - Steam sterilization using fractional vacuum process
 - Steam sterilizer according to DIN EN 285 and validated according to DIN EN ISO 17665
 - Sterilization using fractionated vacuum process at 134 °C/holding time 5 min
- ▶ If several devices are sterilized at the same time in the same steam sterilizer: Ensure that the maximum permitted load according to the manufacturers' specifications is not exceeded.

3.9 Storage

- ▶ Store sterile products in germ-proof packaging, protected from dust, in a dry, dark, temperature-controlled area.

4. Technical service

CAUTION

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

- ▶ For service and repairs, please contact your national B. Braun/Aesculap agency.

Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 7461 95-1601

Fax: +49 7461 16-2887

E-Mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

5. Disposal

WARNING

Risk of infection due to contaminated products!

- ▶ Adhere to national regulations when disposing of or recycling the product, its components and its packaging.

Note

The user institution is obliged to reprocess the product before its disposal, see Validated reprocessing procedure.

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