
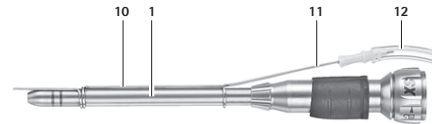
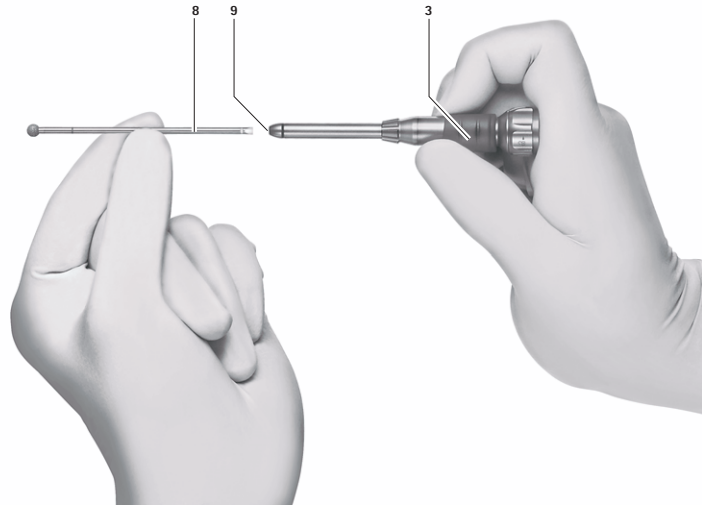
- GB** Instructions for use/Technical description
Hi-Line XS handpiece
- USA** Hi-Line XS-handstycke
- D** Gebrauchsanweisung/Technische Beschreibung
Hi-Line XS-Handstücke
- F** Mode d'emploi/Description technique
Pièces à main Hi-Line XS
- E** Instrucciones de manejo/Descripción técnica
Piezas de mano Hi-Line XS
- I** Istruzioni per l'uso/Descrizione tecnica
Manipoli Hi-Line XS
- P** Instruções de utilização/Descrição técnica
Cabos Hi-Line XS
- NL** Gebruiksaanwijzing/Technische beschrijving
Hi-Line XS-handstuk
- S** Bruksanvisning/Teknisk beskrivning
Hi-Line XS-handstycke
- RUS** Инструкция по применению/Техническое описание
Наконечники Hi-Line XS
- CZ** Návod k použití/Technický popis
Násadce Hi-Line XS
- PL** Instrukcja użytkowania/Opis techniczny
Rękojeści Hi-Line XS
- SK** Návod na používanie/Technický opis
Násady Hi-Line XS
- TR** Kullanım Kılavuzu/Teknik açıklama
Hi-Line XS- el aletleri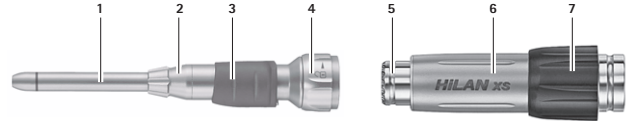

B | BRAUN
SHARING EXPERTISE

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 (0) 7461 95-0 | Fax +49 (0) 7461 95-26 00 | www.aesculap.com

Aesculap – a B. Braun company

TA-Nr. 011267 11/12 V6 Änd.-Nr. 46302

Technical alterations reserved

Legend

- 1 Sheath
- 2 Handpiece
- 3 Slide
- 4 Threaded ring
- 5 Handpiece coupling
- 6 HiLAN XS motor
- 7 Safety stop
- 8 Tool
- 9 Tool coupling
- 10 Spray nozzle
- 11 Tube olive
- 12 Rinsing tube

Symbols on product and packages

	Rotate "right" (clockwise) to unlock the handpiece
	Rotate "left" (counterclockwise) to lock the handpiece
	Short handpiece or tool (one ring)
	Medium handpiece or tool (two rings)
	Long handpiece or tool (three rings)
	Short XL handpiece or tool (one wide ring)
	Medium XL handpiece or tool (two wide rings)
	Long XL handpiece or tool (three wide rings)
 e.g. 2011-02	Reference to the next due maintenance appointment (date) with the international B. Braun/Aesculap agency, see Technical Service

1. Safe handling

CAUTION

Federal law restricts this device to sale by, or on order of a physician!

- ▶ Remove the transport packaging and clean the new product, either manually or mechanically, prior to its initial sterilization.
- ▶ Prior to use, check that the product is in good working order.
- ▶ To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Only combine Aesculap products with each other.
- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- ▶ Keep the instructions for use accessible for the user.
- ▶ Only combine handpieces and tools with matching ring markings.

Single-use product

Infection hazard for patients and/or users and impairment of product functionality due to reuse. Risk of injury, illness or death due to contamination and/or impaired functionality of the product!
▶ Do not reprocess the product.

The products GE280SU to GE298SU, GE401SU to GE729SU and GB761SU to GB766SU are sterilized by radiation and are supplied in sterile packaging.

The product must not be reused.

- ▶ Do not use products from open or damaged sterile packaging.
- ▶ Prior to each use, inspect the product for loose, bent, broken, cracked or fractured components.
- ▶ Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- ▶ Do not re-sterilize the product.
- ▶ Do not use the product after its use-by date.

2. Product description

2.1 Components required for operation

Art. no.	Designation
e.g. GA740R, GA741R, GA519 e.g. GD670, GD675	HiLAN XS motor microspeed uni
GB751R, GB752R, GB753R	Handpieces, straight
GB756R, GB757R, GB758R	Handpieces, angled
GB771R, GB772R, GB773R	Handpieces, angled, XL
GE401R/SU to GE437R/SU, GE456SU to GE475SU GE501R/SU to GE574R/SU GE603R/SU to GE658R/SU GE700SU to GE729R/SU GE280SU to GE288SU GE290SU to GE298SU	Tools for: GB751R, GB756R GB752R, GB757R GB753R, GB758R GB771R GB772R GB773R
GB761SU to GB766SU	Spray nozzles

2.2 Intended use

The straight handpieces GB751R to GB753R and the angled handpieces GB756R to GB758R, GB771R to GB773R are used in micro-, neuro-, spinal, ENT, maxillary and orthopedic surgery.

2.3 Operating principle

The handpieces take up the rotational speed of the motor and transfer the motor rotation to the tool held by the handpiece, at a transmission ratio of 1:1.

The handpieces can be coupled to the motor in 15 different positions.

The tools (e.g. GE401R/SU to GE658R/SU) and the XL tools (e.g. GE280SU to GE298SU and GE700R/SU to GE729R/SU) are used to work on bones.

The cutting wheels (GE430SU, GE530SU) and routers (GE528SU, GE628SU) are used to work on metal (implants, bone screws, etc.) and bones.

3. Preparation

Non-compliance with the following instructions will preclude all responsibility and liability in this respect on the part of Aesculap.

- ▶ When setting up and operating the product, adhere to
 - national regulations for installation and operation,
 - national regulations on fire and explosion protection.

3.1 First use

Risk of injury and/or product malfunction due to incorrect operation of the pneumatic medical system!

- ▶ Adhere to the instructions for use of any medical device.

- ▶ Prior to using the handpieces and their accessories, check for visible damage.
- ▶ Only use handpieces and accessories if they are in perfect condition.

4. Working with the handpieces

4.1 System set-up

Risk of injury and material damage due to inadvertent activation of the motor!

- ▶ Turn the safety stop on the motor to the "0" position.
- ▶ Change tools only with the motor secured by the safety stop.

Risk of infection and contamination! The handpieces are supplied in unsterile condition!

- ▶ Prepare handpieces sterile before putting them into operation.

Risk of injury from bent or snapped tools!

- ▶ Only use handpieces and tools marked with the same number of rings.
- ▶ Do not use tools on which the ring markings are not visible anymore.

Connecting the accessories

Combinations of accessories that are not mentioned in the present instructions for use may only be employed if they are specifically intended for the respective application, and if they do not compromise the performance and safety characteristics of the products.

4.2 Coupling a tool

Risk of burns and other injuries due to bent or incorrectly coupled tools!

- ▶ Only use perfectly straight tools.
- ▶ Make certain that the tool is coupled correctly.
- ▶ Do not use tools on which the ring markings are not visible anymore.

- ▶ To prevent cutting through the glove, be careful when touching the tool where it is sharp.
- ▶ Insert tool **8** into the tool coupling **9** on the handpiece **2** until the pressure point. Do not use tools that get jammed or are difficult to insert.
- ▶ Insert tool **8** into the handpiece **2** as far as it will go, until the correct position is found.
The labeling of the rings on the tool must be covered by the handpiece shaft.
The tool is locked automatically.
- ▶ If the pressure point cannot be passed with the tool: Completely pull back slide **3** against the spring pressure and push the tool into the handpiece **2** down to the stop.
- ▶ Release slide **3**.
The tool must remain fixed in position even when pulled.

4.3 Uncoupling the tool

- ▶ Hold tool 8.
- ▶ Pull back slide 3 at handpiece 2.
- ▶ Remove tool 8 from the handpiece.

4.4 Connecting a handpiece to the motor

- ▶ Turn safety stop 7 on motor 6 to the "0"-position.
 - ▶ Turn threaded ring 4 of handpiece 2 in the direction of the arrow in symbol  as far as it will go.
 - ▶ Install handpiece 2 on the handpiece coupling 5 of the motor 6 in such a way that the pin on the handpiece engages in one of the notches of the handpiece coupling.
 - ▶ Turn threaded ring 4 in the direction of the arrow in symbol  as far as it will go.
 - ▶ Make certain that the threaded ring is fastened to its limit stop, because otherwise it may come loose due to vibrations during operation.
- The handpiece is connected now.

4.5 Disconnecting a handpiece from the motor

- ▶ Turn safety stop 7 on motor 6 to the "0"-position.
- ▶ Turn threaded ring 4 in the direction of the arrow in symbol  as far as it will go.
- ▶ Detach handpiece 2 from the motor.

4.6 Connecting/Disconnecting the spray nozzle

WARNING

Risk of spray nozzle breakage caused by excessive or frequent bending!
▶ Handle the spray nozzle with appropriate care.

Note

A spray nozzle can be attached to the handpieces for the purpose of cooling the tools and, especially, to avoid heat necroses.

If the connection of the spray nozzle at handpiece shaft is unstable, vibrations can cause the tip of the spray nozzle to touch the rotating tool, leading to metal chipping.

Assignment of spray nozzles

Handpieces (art. no.)	Rings	Spray nozzles (art. no.)
GB751R, GB756R	I	GB761SU
GB752R, GB757R	II	GB762SU
GB753R, GB758R	III	GB763SU
GB771R	XLI	GB764SU
GB772R	XLII	GB765SU
GB773R	XLIII	GB766SU

Connecting the spray nozzle

- ▶ From the front, push spray nozzle 10 onto sheath 1 of the handpiece.
- ▶ Fasten irrigation tube 12 onto tube connector 11. To do this, hold spray nozzle 10 at tube olive 11.
- ▶ If necessary, align the tip of spray nozzle towards the tool.
- ▶ Set the irrigation flow at the control unit, as described in the instructions for use of HiLAN irrigation pump GA530 or microspeed uni control unit with cooling unit GD670.

Disconnecting the spray nozzle

- ▶ Pull off irrigation tube 12 from tube olive 11.
- ▶ Remove spray nozzle 10 from shaft 1.

4.7 Function checks

- ▶ Always perform a trial run prior to using the system.
- ▶ Make certain the handpieces and tools are in proper condition.
- ▶ Make certain the handpiece is coupled correctly.
- ▶ Make certain the tool is coupled correctly.
- ▶ Watch out for damage, irregular running noises, excessive vibrations and excessive heat developed by the handpiece.
- ▶ Verify that the handpiece is no hotter than lukewarm after a short period of use.

4.8 Safe operation

WARNING

Risk of injury and/or malfunction!
▶ Always carry out a function check prior to using the product.

WARNING

Risk of injury when using the product beyond the field of view!
▶ Apply the product only under visual control.

WARNING

Risk of injury from bent or snapped tools!
▶ Only use perfectly straight tools. Only apply mild pressure when using a tool.

WARNING

Eye injuries caused by particles coming loose from the tool!
▶ Always wear protective goggles when using the motor system.

WARNING

Risk of burns and necroses from hot cutting edges of the tool!
▶ Tools must be continuously cooled during operation.
▶ Only use sharp tools.

WARNING

Coagulation of patient's tissue caused by a hot handpiece!

- ▶ Allow the handpiece to cool down.
- ▶ Do not use a handpiece whose bearings are damaged.

WARNING

Risk of injury from uncoupled tools!
Risk of burns from a hot fixation screw!
Damage to the fixation screw resulting in loss of functionality due to moving the slide during operation!
▶ Never move the slide on the handpiece while the system is operating.

WARNING

Risk of infection and contamination due to aerosol formation!
▶ Take appropriate protective measures, e.g., watertight protective clothing, face mask, protective goggles, and a suction extraction system.

WARNING

Risk of injury and damage to the tool/system! The rotating tool may catch cover drapes (textile or other)!

- ▶ Never allow the tool to touch cover drapes (textile or other) during operation.

WARNING

Risk of tissue injuries/necrosis when cutting through metals (e.g. implants) using the cutting wheel or the routers!
▶ Cover the operating area.
▶ Carefully remove metal shavings from the operating site.
▶ Make certain that the implant is properly cooled.

CAUTION

Damage to micro instruments caused by incorrect handling!

- ▶ Protect products against overstraining.
- ▶ Do not bend the handpiece shaft.

Note

Do not run the handpiece without a tool coupled for any length of time!

- ▶ To fully exploit the advantages of this power system, operate it at maximum motor speed whenever possible.
- ▶ Remove any surface contamination with a moistened, disposable wipe/paper tissue immediately after use.

5. Validated reprocessing procedure

5.1 General safety instructions

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for sterile processing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note

Mechanical reprocessing should be favored over manual cleaning as it gives better and more reliable results.

Note

Successful processing of this medical device can only be ensured if the processing method is first validated. The operator/sterile processing technician is responsible for this.
The recommended chemistry was used for validation.

Note

If there is no final sterilization, then a virucidal disinfectant must be used.

Note

For the latest information on reprocessing and material compatibility see also the Aesculap extranet at www.extranet.bbraun.com
The validated steam sterilization procedure was carried out in the Aesculap sterile container system.

5.2 Single-use products

WARNING

Risk of infection for patients and/or users and impairment of product functionality due to reuse. Risk of injury, illness or death due to contamination and/or impaired functionality of the product!
▶ Do not reprocess the product!

Art. no.	Designation
GE401SU to GE655SU GE280SU to GE298SU GE700SU to GE729SU	Tools
GB761SU to GB766SU	Spray nozzles

5.3 Reusable products

Reprocessing limit

Reusable products supplied in sterile condition

Art. no.	Designation
GE401R to GE655R GE702R to GE729R	Tools

- ▶ Do not use products from open or damaged sterile packaging.
- ▶ Do not use the product after its use-by date.

Reusable products supplied in an unsterile condition

Art. no.	Designation
GB751R, GB752R, GB753R	Handpieces, straight
GB756R, GB757R, GB758R	Handpieces, angled
GB771R, GB772R, GB773R	Handpieces, angled, XL

- ▶ Carry out sterile processing of the product prior to first use.

5.4 General information

Dried or affixed surgical residues can make cleaning more difficult or ineffective and lead to corrosion. Therefore the time interval between application and processing should not exceed 6 h; also, neither fixating pre-cleaning temperatures >45 °C nor fixating disinfecting agents (active ingredient: aldehydes/alcohols) should be used.

Excessive measures of neutralizing agents or basic cleaners may result in a chemical attack and/or to fading and the laser marking becoming unreadable visually or by machine for stainless steel.

Residues containing chlorine or chlorides e.g. in surgical residues, medicines, saline solutions and in the service water used for cleaning, disinfection and sterilization will cause corrosion damage (pitting, stress corrosion) and result in the destruction of stainless steel products. These must be removed by rinsing thoroughly with demineralized water and then drying.

Additional drying, if necessary.

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Optical changes of materials, e.g. fading or discoloration of titanium or aluminum. For aluminum, the application/process solution only needs to be of pH >8 to cause visible surface changes.
- Material damage such as corrosion, cracks, fracturing, premature aging or swelling.
- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surfaces and could cause corrosion.
- ▶ Further detailed advice on hygienically safe and material-/value-preserving reprocessing can be found at www.a-k-i.org, link to Publications, Red Brochure – Proper maintenance of instruments.

5.5 Preparations at the place of use

- ▶ Remove any visible surgical residues to the extent possible with a damp, lint-free cloth.
- ▶ Transport the dry product in a sealed waste container for cleaning and disinfection within 6 hours.

5.6 Preparation before cleaning

- ▶ Carry out non-fixating/NaCl-free pre-cleaning immediately after use.

5.7 Cleaning/disinfection

Product-specific safety instructions for the reprocessing procedure

DANGER

Fire hazard!

- ▶ Do not use flammable or explosive cleaning or disinfecting solutions.
- ▶ Ensure that no fluids will penetrate the product.

CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents!

- ▶ Only use cleaning/disinfecting agents approved for surface cleaning. Follow the manufacturer's instructions for the respective cleaning/disinfecting agent.

CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!

- ▶ Use cleaning and disinfecting agents according to the manufacturer's instructions which
 - are approved for plastic material and high-grade steel,
- ▶ Observe specifications regarding concentration, temperature and exposure time.
- ▶ Do not exceed the maximum permitted cleaning temperature of 55 °C.

- ▶ Do not clean motors/handpieces by ultrasonic treatment and do not immerse them in any liquids. To avoid the risk of corrosion/malfunctioning, allow any fluid that has entered the product to drain out immediately.

- ▶ Carry out ultrasound cleaning:
 - as an effective mechanical supplement to manual cleaning/disinfecting.
 - as a pre-cleaning procedure for products with encrusted residues, in preparation for mechanical cleaning/disinfecting.
 - as an integrated mechanical support measure for mechanical cleaning/disinfecting.
 - for additional cleaning of products with residues left after mechanical cleaning/disinfecting.

Validated cleaning and disinfection procedure

Validated procedure	Special features	Reference
Manual cleaning and wipe disinfection <ul style="list-style-type: none"> ■ Handpiece 	<ul style="list-style-type: none"> ■ When cleaning products with movable hinges, ensure that these are in an open position and, if applicable, move the joint while cleaning. ■ Drying phase: Use a lint-free cloth or compressed air for medical purposes ■ Ensure that the product is positioned in such a way that water will not enter the product e.g. through coupling interfaces. (Immediately remove any fluid that entered the product inadvertently.) 	Chapter Manual cleaning/disinfecting and subsection: <ul style="list-style-type: none"> ■ Chapter Manual cleaning and wipe disinfecting
Manual cleaning with ultrasound and immersion disinfection <ul style="list-style-type: none"> ■ Tools 	<ul style="list-style-type: none"> ■ Cleaning brush ■ When cleaning products with movable hinges, ensure that these are in an open position and, if applicable, move the joint while cleaning. ■ Drying phase: Use a lint-free cloth or compressed air for medical purposes 	Chapter Manual cleaning/disinfecting and subsection: <ul style="list-style-type: none"> ■ Chapter Manual cleaning with ultrasound and immersion disinfection
Mechanical alkaline cleaning and thermal disinfecting	<ul style="list-style-type: none"> ■ Insert the product in its proper position in the Eccos holder. ■ When using the rinsing device (GB694R), couple the handpiece onto the rinsing device and connect the rinsing device using the silicone tube with the rinsing connector of the rinsing cart (cleaning and disinfection device). ■ The products may only be cleaned mechanically with an appropriate Eccos holder or rinsing device. 	Chapter Mechanical cleaning/disinfecting and subsection: <ul style="list-style-type: none"> ■ Chapter Mechanical alkaline cleaning and thermal disinfecting

5.8 Manual cleaning/disinfecting

- ▶ Prior to manual disinfecting, allow water to drip off for a sufficient length of time to prevent dilution of the disinfecting solution.
- ▶ After manual cleaning/disinfection, check visible surfaces visually for residues.
- ▶ Repeat the cleaning/disinfection process if necessary.

Manual cleaning and wipe disinfecting

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Cleaning	RT (cold)	-	-	D-W	-
II	Drying	RT	-	-	-	-
III	Wipe disinfection	-	>1	-	-	Meliseptol HBV wipes 50 % Propan-1-ol
IV	Final rinse	RT (cold)	0.5	-	FD-W	-
V	Drying	RT	-	-	-	-

D-W: Drinking water
 FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)
 RT: Room temperature

Phase I

- ▶ Clean the product under running faucet water, using a suitable cleaning brush until all visible residues have been removed from the surfaces.
- ▶ Mobilize non-rigid components, such as set screws, links, etc. during cleaning.

Phase II

- ▶ Dry the product in the drying phase with suitable equipment (e.g. cloth, compressed air), see Validated cleaning and disinfection procedure.

Phase III

- ▶ Wipe all surfaces of the product with a single-use disinfectant wipe.

Phase IV

- ▶ After the specified exposure time (at least 1 min), rinse the disinfected surfaces under running FD water.
- ▶ Drain any remaining water fully.

Phase V

- ▶ Dry the product in the drying phase with suitable equipment (e.g. cloth, compressed air), see Validated cleaning and disinfection procedure.

Manual cleaning with ultrasound and immersion disinfection

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Ultrasonic cleaning	RT (cold)	>15	2	D-W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Intermediate rinse	RT (cold)	1	-	D-W	-
III	Disinfection	RT (cold)	15	2	D-W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
IV	Final rinse	RT (cold)	1	-	FD-W	-
V	Drying	RT	-	-	-	-

D-W: Drinking water
 FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)
 RT: Room temperature
 *Recommended: BBraun Stabimed

- ▶ Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- ▶ Clean the product in an ultrasonic cleaning bath (frequency 35 kHz) for at least 15 min. Ensure that all accessible surfaces are immersed and acoustic shadows are avoided.
- ▶ Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- ▶ If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- ▶ Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- ▶ Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- ▶ Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- ▶ Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- ▶ Drain any remaining water fully.

Phase III

- ▶ Fully immerse the product in the disinfectant solution.
- ▶ Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- ▶ Rinse lumens at least five times at the beginning of the exposure time with an appropriate disposable syringe. Ensure that all accessible surfaces are moistened.

Phase IV

- ▶ Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- ▶ Mobilize non-rigid components, such as set screws, joints, etc. during final rinse.
- ▶ Rinse lumens with an appropriate disposable syringe at least five times.
- ▶ Drain any remaining water fully.

Phase V

- ▶ Dry the product in the drying phase with suitable equipment (e.g. cloth, compressed air), see Validated cleaning and disinfection procedure.

5.9 Mechanical cleaning/disinfecting

Note

The cleaning and disinfection device must be of tested and approved effectiveness (e.g. FDA approval or CE mark according to DIN EN ISO 15883).

Note

The cleaning and disinfection device used for processing must be serviced and checked at regular intervals.

Mechanical alkaline cleaning and thermal disinfecting

Machine type: single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemical/Note
I	Prerinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	FD-W	<ul style="list-style-type: none">■ Concentrate, alkaline:<ul style="list-style-type: none">- pH = 13- <5 % anionic surfactant■ 0.5 % working solution<ul style="list-style-type: none">- pH = 11*
III	Intermediate rinse	>10/50	1	FD-W	-
IV	Thermal disin- fecting	90/194	5	FD-W	-
V	Drying	-	-	-	According to the program for cleaning and disinfection device

D-W: Drinking water
FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)

*Recommended: BBraun Helimatic Cleaner alkaline

- ▶ Check visible surfaces for residues after mechanical cleaning/disinfecting.

5.10 Inspection, maintenance and checks

- ▶ Allow the product to cool down to room temperature.
- ▶ Inspect the product after each cleaning and disinfecting cycle to be sure it is: clean, functional, and undamaged.
- ▶ Hi-Line XS handpiece must be sprayed through with Aesculap-STERILIT oil spray GB600 with adapter GB600820 until clear oil emerges from the product.
- ▶ Check the product for any damage, abnormal running noise, overheating or excessive vibration.
- ▶ Inspect tools for broken, damaged or blunt edges.
- ▶ Set aside the product if it is damaged.

5.11 Packaging

- ▶ Always follow the instructions for use of the respective packaging and storage devices (e.g. instructions for use TA009721 for the Aesculap-Eccos storage system).
- ▶ Insert the product in its proper position in the Eccos holder, or put it on a tray in such a way that the product is protected against damage. Ensure that all cutting edges are protected.
- ▶ Pack trays appropriately for the sterilization process (e.g. in Aesculap sterile containers).
- ▶ Ensure that the packaging will prevent a recontamination of the product.

5.12 Steam sterilization

Note

The handpieces must be steam sterilized due to the limited tolerance to high temperature of the materials in them. Do not sterilize the handpieces with hot air or ethylene oxide (ETO).

The handpieces and tools (GE401R to GE729R) can be sterilized in their storage units.

- ▶ Check to ensure that the sterilizing agent will come into contact with all external and internal surfaces (e.g. by opening any valves and faucets).
- ▶ Validated sterilization process
 - Disassemble the product
 - Steam sterilization through fractionated vacuum process
 - Steam sterilizer according to DIN EN 285 and validated according to DIN EN ISO 17665
 - Sterilization using fractionated vacuum process at 134 °C/holding time 5 min
- ▶ When sterilizing several products at the same time in a steam sterilizer, ensure that the maximum load capacity of the steam sterilizer specified by the manufacturer is not exceeded.

5.13 Sterilization for the US market

- Aesculap advises against sterilizing the device by flash sterilization or chemical sterilization.
 - Sterilization may be accomplished by a standard prevacuum cycle in a steam autoclave.
- To achieve a sterility assurance level of 10⁻⁶, Aesculap recommends the following parameters:

Aesculap Orga Tray/Sterile container (perforated bottom) Minimum cycle parameters*			
Sterilization method	Temp.	Time	Minimum drying time
Prevacuum	270 °F/275 °F	4 min	20 min

*Aesculap has validated the above sterilization cycle and has the data on file. The validation was accomplished in an Aesculap sterile container cleared by FDA for the sterilization and storage of these products. Other sterilization cycles may also be suitable, however individuals or hospitals not using the recommended method are advised to validate any alternative method using appropriate laboratory techniques. Use an FDA cleared accessory to maintain sterility after processing, such as a wrap, pouch, etc.

5.14 Storage

- ▶ Store sterile products in germ-proof packaging, protected from dust, in a dry, dark, temperature-controlled area.

6. Maintenance

Risk of burns, necroses and reduced functionality of the tools due to hot blades!
▶ Do not re-sharpen or manipulate the tools.

To ensure reliable operation, the product must be maintained as indicated on the maintenance label.

e.g. 2011-02

For technical service, please contact your national B. Braun/Aesculap agency, see Technical Service.

7. Troubleshooting list

Malfunction	Detection	Cause	Remedy
Handpiece cannot be coupled to the motor	-	Threaded ring not in "unlocked" position	Turn the threaded ring in the direction of the arrow in symbol as far as it will go
Tool cannot be coupled to the handpiece	-	Tool shaft bent	Use a new tool
	-	Handpiece sheath bent	Have product repaired by the manufacturer
Tool cannot be uncoupled	Pressure point cannot be passed	Automatic locking mechanism defective	Couple the tool with the slide actuated
	Tool can not be uncoupled even with the slide fully actuated	Fixation screw worn	Have product repaired by the manufacturer
Handpiece not running	Locking ring/clip in locked position	Slide locked	Rotate the locking ring/locking clip by 180° to unlock the slide
	Motor runs with handpiece decoupled	Handpiece blocked	Have product repaired by the manufacturer
Loud running noise	Motor safety stop in "0" position	Motor locked	Turn the safety stop on the motor to the "I" position
	Loud running noise during normal operation	Worn gear or bearing	Have product repaired by the manufacturer
Handpiece heats up too much	Significant heat-up after short period of operation	Worn gear or bearing	Have product repaired by the manufacturer

8. Technical Service

Risk of injury and/or malfunction!
▶ Do not modify the product.

- ▶ For service and repairs, please contact your national B. Braun/Aesculap agency.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 (7461) 95 -1601
Fax: +49 (7461) 14 -939
E-Mail: ats@aesculap.de

Or in the US:

Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood
MO, 63042 USA
Aesculap Repair Hotline
Phone: +1 (800) 214 -3392
Fax: +1 (314) 895 -4420

Other service addresses can be obtained from the address indicated above.

9. Accessories/Spare parts

Art. no.	Designation	
GB600	Aesculap maintenance oil	
GB600820	STERILIT-Power Systems adapter	
GB473	Eccos storage mounting set	
GB613R	Eccos storage aid for tools	
GB614R/GB611R	Eccos storage aid for handpieces	
GB694R	Rinsing device	
GB761SU	Spray nozzles (for single use):	1 ring
GB762SU		2 rings
GB763SU		3 rings
GB764SU		XLI, 1 ring
GB765SU		XLI, 2 rings
GB766SU		XLI, 3 rings
GE401R to GE655R	Tools:	reusable
GE401SU to GE658SU		single use
GE702R to GE729R	XL tools:	reusable
GE280SU to GE298SU GE700SU to GE729SU		single use
GB770	Locking clip	
JF223R, JF222R	Tray	
JK400, JK401	Container	

10. Technical data

Handpiece	Weight	Marking	Length of shaft	Dimension x Length (mm)
GB751R S74932	30 g	1 ring	38 mm	∅ 17.5 x 91.5
GB752R S74933	35 g	2 rings	68 mm	∅ 17.5 x 121.5
GB753R S74934	40 g	3 rings	98 mm	∅ 17.5 x 151.5
GB756R	35 g	1 ring	38 mm	∅ 17.5 x 108.5
GB757R	40 g	2 rings	68 mm	∅ 17.5 x 138.5
GB758R	45 g	3 rings	98 mm	∅ 17.5 x 168.5
GB771R	54 g	1 ring	128 mm	∅ 17.5 x 198.5
GB772R	58 g	2 rings	158 mm	∅ 17.5 x 228.5
GB773R	68 g	3 rings	218 mm	∅ 17.5 x 288.5

11. Disposal

- Detailed information concerning the disposal of the product is available through your national B. Braun/Aesculap agency, see Technical Service.

12. Distributor in the US/Contact in Canada for product information and complaints

Aesculap Inc.
3773 Corporate Parkway
Center Valley, PA, 18034,
USA

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