EASYMIX® shaker

Instructions for Use

en

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de

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fr

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it

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es

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pt

Brugsanvisning

da

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tr

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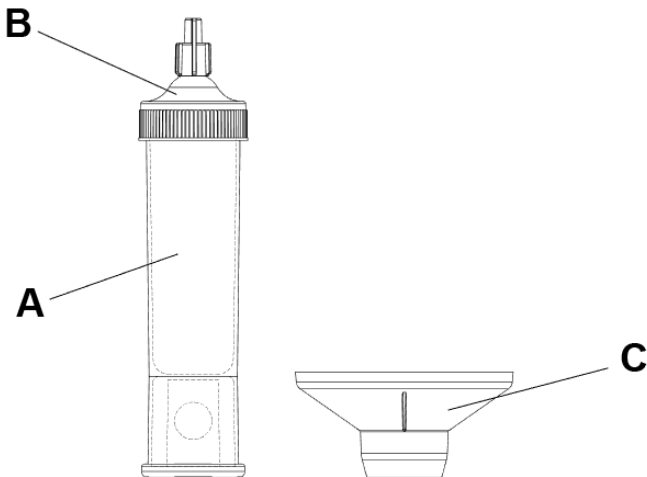
web www.osartis.de

For internal use

EASYMIX® shaker
Instructions for Use
To the Medical Staff's attention

en

Description

Sterile and single-use device for the mixing of acrylic resin components (powder and liquid) and transfer into syringes. The device is composed of a plastic container (A) with a luer-lock cap (B) and a funnel (C).

Intended Use

Mixing of acrylic resin components and transfer into syringes.

Precautions for use

Use for mixing acrylic resins with powder weight up to 24g and 10ml liquid volume. The mixed acrylic resin must be suitable to be drawn up into a syringe. Please refer to the manufacturers information of the acrylic resin.

Instructions for use

Set the plastic container (A) on a flat surface and open the luer-lock cap (B). Pour the liquid component into the container. A funnel (C) is provided to ease the process of filling the powder into the container (A). The funnel has to be placed onto the open end of the plastic container. Pour the powder component into the container and close the luer-lock cap firmly. Manually shake the container forcefully for at least 30 seconds (approximately 2 strokes per second) until a homogenous acrylic resin

mass is achieved. The mixing phase can be extended if necessary. Please refer to the manufacturers information of the acrylic resin. Open the luer-lock connection by removing the plug and connect the syringe with luer-lock of the plastic container cap. Turn the container upside down and collect the acrylic resin at the cap side. Draw up the acrylic resin into the syringe. Remove the syringe from the luer-lock connection and fill additional syringes in the same manner. Caution: At low viscosity it can be needed to turn the container in horizontal position temporarily to stop spilling out of the acrylic resin! For filling of the syringes the cartridge has to be positioned upside down.

Warnings

Do not use with bone void fillers that contain solvents of PP (Polypropylene).

The sterility is assured only if the unit container is not damaged or opened.

The device is single-use intended for use on a single patient. Do not re-sterilise and/or re-use any of the components. Re-use should not be carried out since it can cause risk of infection for the patient and the maintenance of the same performances is not assured. Moreover a re-sterilization may alter the morphology and/or chemical/physical/mechanical properties causing lastly a malfunction of the same with serious health risks for the patient.

Any residual material must be considered surgical waste and therefore it must be eliminated at the end of the surgical procedure.

Disposal

Disposal of the device should be in accordance with local waste regulations.

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