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IMPLANT COMPONENT:	Cancellous bone screw Ø 4.0 mm
SYSTEM:	ADDplus™ anterior distraction device plus osmium™ anterior cervical plate system
LANGUAGE:	ENGLISH The only version applicable to users in the U.S. is the version intended especially for the United States.
	BEFORE CLINICAL USE, TAKE NOTE OF THESE INSTRUCTIONS FOR USE AND THE SYSTEM-RELATED SURGICAL TECHNIQUE!

GENERAL INFORMATION

Not all products are released for sale in all countries. If you require detailed information on this, please contact us.

Please ensure that you are using and observing the most current instructions for use and surgical technique of this system at all times. These can be downloaded any time free of charge at: www.ifu.ulrichmedical.com

SPECIAL WARNING AND APPLICATION INSTRUCTIONS

Cancellous bone screws are implant components for the following ulrich medical implant systems:

- ADDplus | anterior distraction device plus
- osmium | anterior cervical plate system

Cancellous bone screws serve the conventional screw fixation of the aforementioned implant systems to adjacent vertebral bodies.

Cancellous bone screws have a diameter of 4.0 mm and are available in eight lengths (12 - 26 mm).

When using cancellous bone screws with the ADDplus implant system, please refer to the ADDplus user guide (WS 2295-01).

When using cancellous bone screws with the osmium implant system, please refer to the osmium user guide (WS 1395).

Before use, carefully check the protective packaging for damage which could impair the sterility of the product.

MATERIAL INFORMATION

The ulrich medical cancellous bone screws are manufactured from a titanium alloy according to ISO 5832-3; ASTM F136.

CLEANING, DISINFECTION AND STERILIZATION

For the cleaning, disinfection and sterilization of unsterilized implants and instruments, we refer to UH 1100 "Processing manual implants and instruments".

For individual instruments which are correspondingly marked in the system-related surgical technique (⚠), we refer to the "Assembly and disassembly instructions with special cleaning instructions". You can download them any time free of charge at: www.ifu.ulrichmedical.com

IMPLANT COMPONENT:	Cancellous bone screw Ø 4.0 mm, length xx
SYSTEM:	ADDplus™ anterior distraction device plus osmium™ anterior cervical plate system
COUNTRY:	USA
Rx only	BEFORE CLINICAL USE, TAKE NOTE OF THESE INSTRUCTIONS FOR USE AND THE SYSTEM-RELATED SURGICAL TECHNIQUE!

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

MANUFACTURER:
ulrich GmbH & Co. KG
Buchbrunnweg 12
89081 Ulm | Germany

DISTRIBUTED IN THE USA BY:
ulrich medical USA Inc.
18221 Edison Avenue
Chesterfield, MO 63005 | USA
(636) 519-0268

Package Insert: IMPORTANT INFORMATION for Surgeons, Medical Staff and Patients on the Use of the cancellous bone screw.

FOR QUESTIONS OR FURTHER INFORMATION

For questions or further information or to obtain a copy of the AddPlus Surgical Technique Manual, please contact ulrich medical USA, located at 612 Trade Center Boulevard, Chesterfield, MO 63005, or call (636) 519-0268 or send an email to spine@ulrichmedicalusa.com.

Not all products are released for sale in all countries. If you require detailed information on this, please contact us.

Cancellous bone screws are implant components for the following ulrich medical implant system:

- AddPlus | anterior distraction device plus

Cancellous bone screws serve the conventional screw fixation of the aforementioned implant systems to adjacent vertebral bodies.

Cancellous bone screws have a diameter of 4.0 mm and are available in eight lengths (12 - 26 mm).

When using cancellous bone screws with the AddPlus implant system, please refer to the AddPlus user guide (WS 2295-01).

MATERIAL INFORMATION

ulrich medical cancellous bone are made of a titanium alloy in accordance with ISO 5832-3; ASTM F136.

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