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IMPLANT COMPONENT:	osmium™ screw Ø 5.0 mm, incl. bolt
SYSTEM:	ADDplus™ anterior distraction device plus osmium™ anterior cervical plate system
LANGUAGE:	ENGLISH The only version applicable to users in the U.S. is the version intended especially for the United States.
	BEFORE CLINICAL USE, TAKE NOTE OF THESE INSTRUCTIONS FOR USE AND THE SYSTEM-RELATED SURGICAL TECHNIQUE!

GENERAL INFORMATION

Not all products are released for sale in all countries. If you require detailed information on this, please contact us.
Please ensure that you are using and observing the most current instructions for use and surgical technique of this system at all times. These can be downloaded any time free of charge at: www.ifu.ulrichmedical.com

SPECIAL WARNING AND APPLICATION INSTRUCTIONS

osmium screws are implant components for the following ulrich medical implant systems:

- ADDplus | anterior distraction device plus
- osmium | anterior cervical plate system

osmium screws serve the monocortical fixation of the aforementioned implant systems in the adjacent vertebral bodies with reduced bone quality.

osmium screws have a diameter of 5.0 mm and are available in three lengths (14, 16, 18 mm).

When using osmium screws with the ADDplus implant system, please refer to the ADDplus Instructions for Use (WS 2295-01).

When using the osmium screws with the osmium implant system, please refer to the osmium Instructions for Use (WS 1395).

Before use, carefully check the protective packaging for damage which could impair the sterility of the product.

MATERIAL INFORMATION

The ulrich medical osmium screws are manufactured from a titanium alloy according to ISO 5832-3; ASTM F136.

CLEANING, DISINFECTION AND STERILIZATION

For the cleaning, disinfection and sterilization of unsterilized implants and instruments, we refer to UH 1100 "Processing manual implants and instruments".

For individual instruments which are correspondingly marked in the system-related surgical technique (Δ), we refer to the "Assembly and disassembly instructions with special cleaning instructions". You can download them any time free of charge at: www.ifu.ulrichmedical.com

IMPLANT COMPONENT:	osmium™ screw Ø 5.0 mm, incl. bolt, length xx
SYSTEM:	AddPlus™ anterior distraction device plus
COUNTRY:	USA
Rx only	BEFORE CLINICAL USE, TAKE NOTE OF THESE INSTRUCTIONS FOR USE AND THE SYSTEM-RELATED SURGICAL TECHNIQUE!

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

MANUFACTURER:
Ulrich GmbH & Co. KG
Buchbrunnweg 12 | 89081 Ulm | Germany

DISTRIBUTED IN THE USA BY:
Ulrich Medical USA Inc.
18221 Edison Avenue | Chesterfield, MO 63005 | USA
(636) 519-0268

Package Insert: IMPORTANT INFORMATION for Surgeons, Medical Staff and Patients on the Use of the osmium screw.

FOR QUESTIONS OR FURTHER INFORMATION

For questions or further information or to obtain a copy of the AddPlus Surgical Technique Manual, please contact ulrich medical USA, located at 18221 Edison Avenue | Chesterfield, MO 63005, or call (636) 519-0268 or send an email to spine@ulrichmedicalusa.com.

Not all products are released for sale in all countries. If you require detailed information on this, please contact us.

osmium screws are implant components for the following ulrich medical implant system:

- AddPlus | anterior distraction device plus

osmium screws serve the monocortical fixation of the aforementioned implant systems in the adjacent vertebral bodies with reduced bone quality.

osmium screws have a diameter of 5.0 mm and are available in three lengths (14, 16, 18 mm).

When using osmium screws with the AddPlus implant system, please refer to the AddPlus user guide (WS 2295-01).

MATERIAL INFORMATION

ulrich medical osmium screws are made of a titanium alloy in accordance with ISO 5832-3; ASTM F136.

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